

Mrx Transport Monitor/Defibrillator Check List

Unit/Location: _____

Week of: ___/___/___ to ___/___/___

Device Number: _____

Procedure to Perform MRx Defib Test

1. Unplug Defib Pads from Therapy Cable
2. Plug Therapy Cable into Test Load
3. Power on Defibrillator/Monitor
4. Deliver 150J Shock into Test Load
5. Verify energy delivered is in range
6. Turn Off Defibrillator/Monitor
7. Re-connect Defib Pads to Therapy Cable

Date	Initials	By: _____	By: _____	By: _____	By: _____	By: _____	By: _____
------	----------	-----------	-----------	-----------	-----------	-----------	-----------

RFU (Ready For Use) Indicator shows hourglass - If red "X" displayed, turn unit on and allow self-check to complete. If red "X" remains visible, call Service Center for Replacement and take unit out of service							
Therapy Cable Present							
Defibrillation Test Performed							
Pre-connect Defibrillation Pads Present (2 sets)							
3 - Lead Monitor Cable Present							
Monitoring Electrodes Present (at least 6)							
Blood Pressure Cuff and Tubing Present & Connected							
Pulse Oxyemtry Cable/Probe Present							
Filterline H Set (End-Tidal CO2 Airway Adapters)							
Pulse Oxyemtry Cable/Probe Present							
QCPR Device Present							
QCPR Adhesive Pads Present							
Test Load Present							
Deviced Plugged in to Electrical Outlet for Charging							

Signature	Initials	Signature	Initials
_____	_____	_____	_____
_____	_____	_____	_____
_____	_____	_____	_____
_____	_____	_____	_____

Notes:
